

/ or friction, it will show up as a white glowing area on the participant hands.

To demonstrate cross contamination using GlitterBug® Powder :-

1. Sprinkle some powder onto a piece of paper;
2. Pretend to sneeze on the paper;
3. Ask people to take the paper and sign their name for attendance records;
4. Using the disclosure center look at the fingers of the first person taking the paper. The glowing area demonstrates the cross contamination of 'bugs' from the paper to their hands.



The GlitterBug® Deluxe kit includes:-

- GlitterBug Disclosure Center
- Carrying Bag
- GlowBar lamp with 4 D-cell batteries
- AC/DC adapter for plug-in convenience
- Hand washing DVD
- GlitterBug® Potions (240ml)
- GlitterBug® Powder (70g)
- GlitterBug® Instruction Manual
- How to Wash Poster
- Two sheets of "I Was Rude To a Germ" stickers
- Two Hand Brushes
- GlitterBug® Magnet
- GlitterBug® Champion Medallion.

Note: Individual items included in kits are not returnable

Ordering Details

Deluxe GlitterBug® Kit

Product Code: BRDELUXEKIT